HEPES PLUS

DEUTSCH

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ENGLISH

¹HEPES PLUS is a ready to use HEPES buffered Human Tubular Fluid (HTF) Medium with Human Serum Albumin (HSA) used for In Vitro Fertilization (IVF) procedures that are achieved outside the CO₂ incubator, including the retrieval, handling, micromanipulation and transfer of human gametes and embryos.

COMPOSITION

- HEPES HTF
- Gentamicin
- Bicarbonate
- Ultra Pure Water
- Amino Acids
- Glucose
- Human Serum Albumin
- Physiological Salts
- Lactate

PRODUCT SPECIFICATIONS

- pH: between 7.00 - 7.40
- Sterility: sterile (SAL 10⁻³)
- Mouse Embryo Assay (Blastocysts after 96h): ≥ 80%
- A certificate of analysis and MSDS are available upon request.
- Endotoxins: < 0.25 EU/ml

STABILE A Sterilized using aseptic processing techniques.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if any discoloration, turbidity or precipitation is noticed.
- Do not use the product if seal of the container is defect or opened upon delivery.

INSTRUCTIONS FOR USE

- 1 Remove the desired volume of product using aseptic procedures.
- 2 Write the date of the first opening on the product label.
- 3 Once removed, do not return any volume of product into the original container.
- 4 Warm ¹HEPES PLUS to 37°C prior to use.

STORAGE AND STABILITY

- Store ¹HEPES PLUS between 2°C to 8°C.
- Do not freeze or expose to temperatures higher than 39°C.
- The product is stable until the expiration date shown on the label or within 7 days of the date of first opening, provided that the proper aseptic procedures have been achieved by the user.

WARNING AND PRECAUTIONS

VITROMED recommends handling of media only with aseptic techniques. Best measures to prevent infections resulting from use of medical products prepared from human blood or plasma include precise screening of each donation and plasmapools for distinct markers of infection and the use of validated methods for inactivation of viruses during production. Despite this, a possible transmission cannot completely ruled out. To the current state, there are no reports about confirmed cases about transmissions of viral infections or Creutzfeldt-Jakob disease (CJD) due to human albumin. Always wear protective clothing when handling specimens. Always work under strict hygienic conditions (LAF-bench ISO class 5) to avoid contamination. ¹HEPES PLUS contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic. Ensure a correct disposal.

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Stück pro VE
Piece per unit

100 ml PET-Flasche
100 ml PEI bottle

Technischer Support
For technical support

VITROMED
Germany

VITROMED GmbH
Wildenbruchstrasse 15
07745 Jena
Germany

Phone +49 36 41 5 39 19 76
E-Mail support@vitromed.com
Web www.vitromed.com

Stand der Informationen
Date of Information

01.05.2017