HYLASE

DEUTSCH

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ENGLISH

HYLASE is a ready-to-use media for the denudation of oocytes prior to the intracellular microinjection (ICSI). It contains 80 IU/ml of Hyaluronidase in HEPES buffered Human Tubul Fluid (HTF) Medium with Human Serum Albumin (HSA). Short exposure of the oocytes with Hyaluronidase is breaking the hyaluronic acid bonds between the cumulus cells and makes it easy to remove the cumulus during the denudation process.

COMPOSITION

- HEPES HTF
- Human Serum Albumin
- 80 IU/ml Hyaluronidase
- Gentamicin

PRODUCT SPECIFICATIONS

- pH: between 7.20 - 7.40
- Sterility: sterile (SAL 10-3)
- Mouse Embryo Assay (Blastocysts after 96h): $\geq 80\%$
- A certificate of analysis and MSDS are available upon request.
- Endotoxins: < 1.00 EU/ml
- Osmolality: 257 - 273m Osm/kg

STIMULA Sterilized using aseptic processing techniques.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if any discoloration, turbidity or precipitation is noticed.
- Do not use the product if seal of the container is defect or opened upon delivery.

INSTRUCTIONS FOR USE

- 1 Prepare a petri dish containing one drop **HYLASE** and several drops of **HEPES PLUS** (80 μ l to 100 μ l each) and cover with **OIL**. Warm the dish up to 37°C.
- 2 Transfer 3 to 6 cumulus oocyte complexes (COC) into the **HYLASE** droplet and gently pipette the COCs up and down using a denuding pipette (**DENUPET** 300 μ m) to remove the oocytes from the cumulus.

- 3 After a maximum of 20 to 30 seconds transfer the partially denuded oocytes with a fine denuding pipette (**DENUPET** 140 μ m) into a **HEPES PLUS** droplet and remove the remaining granulosa cells by further denudation.
- 4 Wash the oocytes thoroughly in the remaining **HEPES PLUS** droplets.
- 5 The oocytes are now ready for the ICSI procedure.

STORAGE AND STABILITY

- Store **HYLASE** between 2°C to 8°C.
- Do not freeze before use.
- Leave product in original packaging and protect from light.
- The product is stable until the expiration date shown on the label or within 7 days of the date of first opening, provided that the proper aseptic procedures have been achieved by the user.

WARNING AND PRECAUTIONS

VITROMED recommends handling of media only with aseptic techniques. Best measures to prevent infections resulting from use of medical products prepared from human blood or plasma include precise screening of each donation and plasmapools for distinct markers of infection and the use of validated methods for inactivation of viruses during production. Despite this, a possible transmission cannot completely ruled out. To the current state, there are no reports about confirmed cases about transmissions of viral infections or Creutzfeldt-Jakob disease (CJD) due to human albumin. Always wear protective clothing when handling specimens. Always work under strict hygienic conditions (LAF-bench ISO class 5) to avoid contamination. **HYLASE** contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic. Ensure a correct disposal.

DE / EN

Artikelbezeichnung Name of product	HYLASE
REF	V-HYA
Stück pro VE Piece per unit	5 x 1,0 ml Phiole 5 x 1.0 ml vial

Technischer Support
For technical support

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VITROMED GmbH