YOIL

DEUTSCH

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ENGLISH

YOIL is pre-washed, light paraffin oil for use in IVF-, ICSI- procedures and ready to use. YOIL prevents the evaporation of water and protects the media against changes in osmolality and pH. Culture of embryos in small volumes of medium under oil helps to enhance rates of embryonic development. YOIL overlay reduces changes in pH caused by loss of CO₂ from the medium when culture dishes are taken out of the incubator for embryo assessment or handling.

COMPOSITION

• Paraffin Oil

PRODUCT SPECIFICATIONS

- Endotoxins: < 0.25 EU/ml
- Sterility: sterile (SAL 10⁻³)
- Density: 0.83 - 0.86 g/ml
- Viscosity: < 30 cP at 30°C
- Peroxide Value [POV]: < 0.1 mEq/kg
- Mouse Embryo Assay (Blastocysts after 120 h): ≥ 80%
- Use of pharmaceutical grade mineral (paraffin) oil
- A certificate of analysis and MSDS are available upon request.

SYMBOLE A Sterilized using aseptic processing techniques.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if any discoloration, turbidity or precipitation is noticed.
- Do not use the product if seal of the container is defect or opened upon delivery.

INSTRUCTIONS FOR USE

- 1 Since YOIL is pre-washed, no further preparation is necessary.
- 2 Place a bottle of YOIL in an incubator at 37°C 5 hours prior to use (with lid closed).
- 3 YOIL can be used together with all VITROMEDIA products.
- 4 YOIL should be used only for its intended purpose by qualified and trained persons.

STORAGE AND STABILITY

- Store YOIL between 2°C to 8°C.
- Do not freeze before use.
- Leave product in original packaging and protect from light.
- YOIL can be used safely for at least 28 days after opening the container, provided that the proper aseptic procedures have been achieved by the user.
- Do not use after expiry date. Keep away from sunlight.
- The oil may appear cloudy when removed from the refrigerator or at room temperature after extended periods of storage time. Such cloudiness usually disappears when the oil is equilibrated in the incubator for several hours. If it does not disappear discard the product.

WARNING AND PRECAUTIONS

VITROMED recommends handling of media only with aseptic techniques. All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling specimens. Always work under strict hygienic conditions (LAF-bench ISO class 5) to avoid contamination. Ensure a correct disposal.

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Technischer Support
For technical support

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VITROMED GmbH