SPERM

DEUTSCH

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ENGLISH

^YSPERM WASH is a HEPES buffered Human Tubal Fluid (HTF) Medium with Human Serum Albumin (HSA), intended for sperm washing and incubation for Intra Uterine Insemination (IUI), In Vitro Fertilization (IVF) and other Assisted Reproductive Techniques (ART).

COMPOSITION

- HEPES HTF
- Sodium Hyaluronate
- Human Serum Albumin
- Gentamicin
- Physiological Salts

PRODUCT SPECIFICATIONS

- pH: between 7.00 - 7.40
- Endotoxins: < 0.25 EU/ml
- Sterility: sterile (SAL 10⁻³)
- Human Sperm Survival Assay tested
- Mouse Embryo Assay (Blastocysts after 96h): ≥ 80%
- A certificate of analysis and MSDS are available upon request.

STERILE A Sterilized using aseptic processing techniques.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if any discoloration, turbidity or precipitation is noticed.
- Do not use the product if seal of the container is defect or opened upon delivery.

INSTRUCTIONS FOR USE

- 1 Warm the ^YSPERM WASH to 37°C.
- 2 Carefully layer 1.5 ml ^YSPERM WASH on 1 mL of liquefied semen in a centrifuge tube. Place this tube at a 45° angle and incubate for 1 hour at 37°C.
- 3 Gently bring the centrifuge tube to the upright position and aspirate the uppermost 1 mL and place it in a new clean centrifuge tube.
- 4 Add 5 mL of ^YSPERM WASH and centrifuge 10 minutes at 300 x g. Remove supernatant and finally resuspend in 0.4 mL of ^YSPERM WASH.
- 5 The sample is now ready for analysis or use.

STORAGE AND STABILITY

- Store ^YSPERM WASH between 2°C to 8°C.
- Do not freeze before use.
- Leave product in original packaging and protect from light.
- The product is stable until the expiration date shown on the label or within 7 days of the date of first opening, provided that the proper aseptic procedures have been achieved by the user.

WARNING AND PRECAUTIONS

VITROMED recommends handling of media only with aseptic techniques. Best measures to prevent infections resulting from use of medical products prepared from human blood or plasma include precise screening of each donation and plasmapools for distinct markers of infection and the use of validated methods for inactivation of viruses during production. Despite this, a possible transmission cannot completely ruled out. To the current state, there are no reports about confirmed cases about transmissions of viral infections or Creutzfeldt-Jakob disease (CJD) due to human albumin. Always wear protective clothing when handling specimens. Always work under strict hygienic conditions (LAF-bench ISO class 5) to avoid contamination. ^YSPERM WASH contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic. Ensure a correct disposal.

DE / EN

Artikelbezeichnung Name of product	SPERM WASH
REF	V-SWM-100
Stück pro VE Piece per unit	100 ml PET-Flasche 100 mL PET bottle

Technischer Support
For technical support

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VITROMED GmbH